



Cleaning instructions for Cattani aspiration systems.

For your personal health and safety, always wear eye, mouth and hand protection when handling this product and before starting cleaning procedures.

Daily

Clean the system using the 'Pulse Cleaner' provided (part No 040720) Using only Puli-Jet Plus* (part No 040750) shake the bottle first, remove the lid and place on a flat surface and squeeze gently until the liquid fills the measure, release the bottle, this will draw back the excess Puli-Jet. Pour one measure into the 'Pulse Cleaner' fill with **warm water (50°C max)** to the top level marked on the inside of the 'Pulse Cleaner',

Weekly

To disinfect the system clean as above but use two measures of 'Puli-Jet Plus'

With the unit switched on, place the Pulse Cleaner on to a solid surface and firmly connect the large and small aspirator terminals with the sliders open, onto the pulse cleaner. This will start the cleaning process. When the liquid is completely aspirated remove the tips and replace into the support. The residual cleaner should be poured down the spittoon to clean the internal pipes and Spittoon valve if fitted. Remove the spittoon valve filter, clean and replace. Remove the filter from the tip support and clear any debris, clean the filters in an ultra-sonic cleaner.

Care should be taken to clean around the threads of the body and cap with a tissue removing any residual amalgam dust. Insert two Antifoaming and disinfecting tablets (part No 040825) into the filter, replace filter and cap.

Do not rinse the aspirator or spittoon after cleaning, as Puli-Jet Plus will carry on disinfecting overnight.

Strip down and clean the terminals, spray with Cattani Lubri-Jet (part No 040735) on re-assembly, the suction terminals may be autoclaved at 134°C

Use only Cattani Eco-Jet spray (part No 0407450) or (Eco-Jet wipes part No 070748) as a surface disinfectant for final cleaning. Cattani Eco-Jet is especially formulated not to affect Cattani plastic components.

Weekly.

Remove Tip support tubes and cold sterilize over the weekend

Remove the filter from the Turbo-Jet, tip out any debris and clean the filter in an ultrasonic cleaner. Clean around the threads of the body and cap with a tissue, replace the filter and cap.

ONLY CATTANI CLEANSING PRODUCTS SHOULD BE USED ON THIS SYSTEM.
The use of other manufacturers cleaning products will invalidate the warranty.

Any queries please call Cattani Esam UK Limited on 0044 (0)1527 877 997 and ask for customer care or e-mail info@cattaniesam.co.uk

Cdcleaning1